making work easy

DE

EN

FR

IT

ES

PT

TR

RU

ZH

JA

KO



AR

SILENT compact

Quick Start Guide

Made in Germany

21-6816 15102020

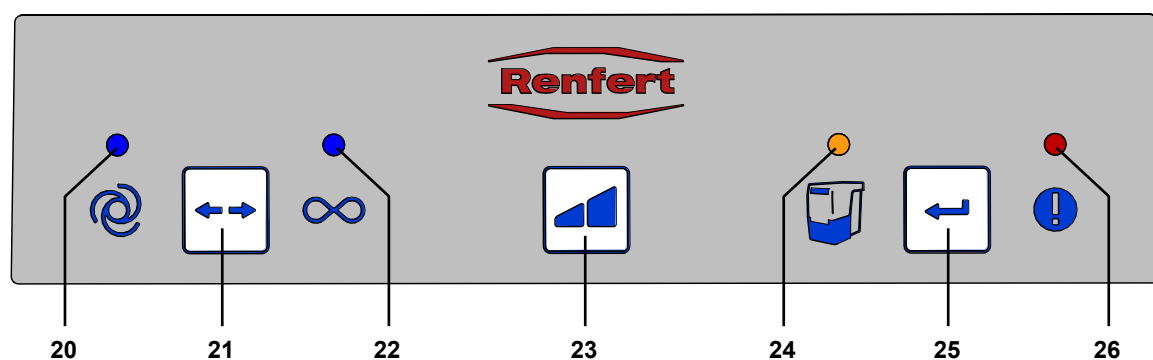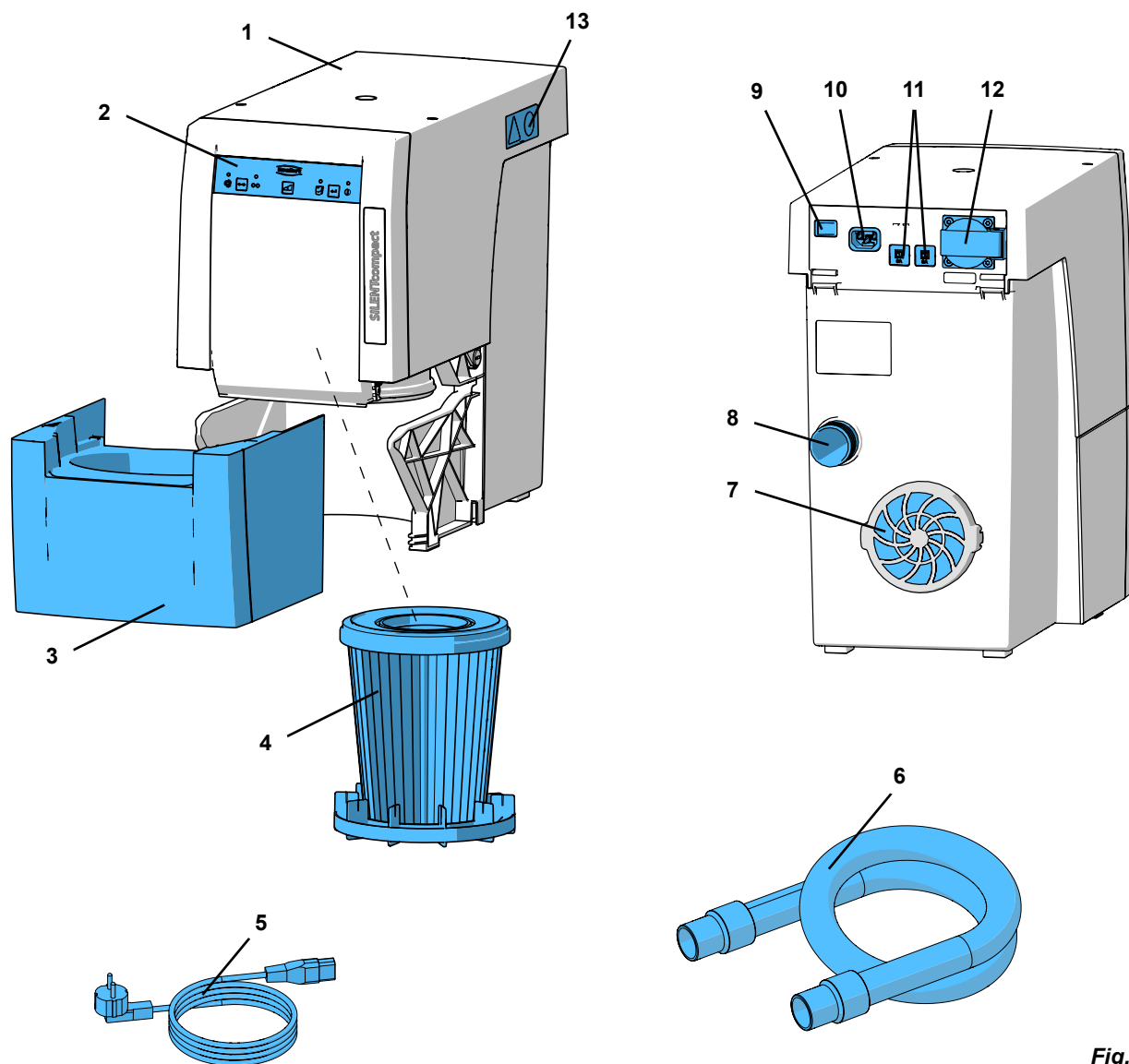Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the product and relevant safety information.

For detailed information about the device, see the instructions for use which you can download from our website under www.renfert.com/p915.
Enter the following item number: 29340000

EN

Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

There is a FAQ video available on this topic in our video gallery under www.renfert.com/p49.

Other symbols are explained as they occur.

Safety

Intended Use

This device is designed to extract dry, non-explosive dust.

The unit is intended solely for use in a commercial dental laboratory and dental practice.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

Improper Use

The equipment is not suitable for use with dental CAM units! For information concerning SILENT extraction units for CAM machines, please refer to www.renfert.com.

Fire-promoting, easily flammable, red-hot, burning or explosive materials must not be suctioned into the device. It is not permitted to suction liquids. This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device. The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

Hazard and Warning Information

General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.

- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.
- ▶ Observe the accident prevention regulations by the trade association!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technologies).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Specific Information

- ▶ The power socket on the device may only be used for the purpose specified in the instructions for use. The connection of other electronic devices could lead to material damage.
- ▶ Before connecting an electronic device to the power socket, ensure that the electronic device is first switched off.
- ▶ Read the operating instructions of the other appliance and comply with the safety instructions contained in the document.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Always refer to the relevant safety data sheets, when extracting hazardous materials.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract flammable or explosive gasses, fumes or dust.
- ▶ The following applications involve increased risks and are therefore not permitted:
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e. unmixed with other dental dusts such as gypsum, resins) can result in spontaneous ignition or the milling dust exploding or printing powder because of a potential exothermic reaction (e.g. due to oxidation).
An alternative cleaning method must be selected for the following milling or printing media in particular (such as manual cleaning):
 - Wood
 - Titanium/titanium-aluminum
 - Light metals and light metal alloys (e.g. aluminum, magnesium)
 - Cobalt chrome powder (e.g. for use in SLM systems)
- If large quantities of light metals such as titanium alloy are processed (e.g. with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.
- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ If the dust extractor is employed to extract hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.
- ▶ Dispose of extracted material according to local statutory regulations.

Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Product Description

Components and Functional Elements

See Fig. 1 (on inside cover)

- | | |
|---|-----------------------------|
| 1 SILENT compact | 8 Suction port |
| 2 Key-pad | 9 On / Off switch |
| 3 Dust drawer | 10 Power supply |
| 4 Fine filter | 11 Device protection switch |
| 5 Mains cable | 12 Unit power socket |
| 6 Suction hose | 13 Service flap |
| 7 Exhaust air filter / exhaust air outlet | |

See Fig. 2 (on inside cover)

- | | |
|--|---------------------------------|
| 20 Display automatic operation | 23 Suction level key |
| 21 Operating mode key, automatic, continuous operation | 24 Display empty suction drawer |
| 22 Display continuous operation | 25 Enter key, save input |
| | 26 Display error message |

Setting Up

Setup

The extraction unit is a free standing appliance which must not be operated in a lying position.

If the device is to be installed in a cabinet, an opening for the exhaust air with the following dimensions must be provided, in addition to the openings for the suction hoses:

- Circular opening: Min. 120 mm diameters.
- Rectangular opening: Min. 170 x 65 mm.

Connection

- ⇒ Insert the suction tube (6) into the suction port (8).
- ⇒ Connect the suction tube to the relevant suction point.
- ⇒ If necessary, shorten the suction tube.

Caution, risk of injury!

When shortening the suction tube, please ensure that the integrated wire is cut as straight as possible.

If the diameter size does not correspond, please use an adapter to prevent a loss in suction performance.

Long suction hoses, tight bends and kinks will considerably reduce the extraction force at the extraction point.

- ⇒ Connect the electronic device to the power socket (12) at the back.

When connecting an electronic device to the extraction unit, please ensure that the maximum permissible capacity for connected devices is not exceeded (Technical data).

- ⇒ Switch the device OFF at the On / Off switch (9).
- ⇒ Connect the power cable (5) to the power supply (10).
- ⇒ Insert the power plug into the building installed wall socket.

Operation

Switching the Unit On

The extractor is switched ON and OFF at the On / Off switch (9).

When the unit is switched on:

- ♦ All 4 displays light up (the display carries out a functional control),
- ♦ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

Select Operating Mode: automatic / continuous operating mode

- Automatic operating mode (20):
In order to function, the extraction unit is dependent on an electronic device connected to the power socket (12).
 - Continuous operating mode (22):
The extraction unit runs continuously.
- ⇒ Press the operating mode key (21).
- ♦ To change the operating mode.

If the suction to a connected electrical appliance (e.g. a handpiece) does not react as required (e.g. does not activate, even though a connected device is in operation) then the switch-on threshold for automatic operation must be adjusted. This is also possible whilst the suction is on.

Suction Level

- ⇒ Press the key suction level (23).
- ♦ Change between the two suction levels.
- The extraction unit always starts with the last set suction level.

Cleaning / Maintenance

Opening the device, other than for the processes described below, is not permissible!

Cleaning

Use a damp cloth to clean the outside of the unit.
Do not use abrasive or solvent-based cleaning agents.

Empty Dust Drawer

After the set time period has expired a signal to empty the dust drawer will occur (factory settings 50 h). Before the dust drawer is emptied, a filter cleaning function is carried out so that loose dust particles fall into the dust drawer.

The time period has expired:

- ♦ An acoustic signal will occur 3 times.
- ♦ The display dust drawer (24) lights up.

When the suction has stopped, an acoustic signal occurs 3 times and a filter cleaning function is carried out.

When the filter cleaning function has finished:

- ⇒ Pull the dust drawer (3) out to the front and empty.
⇒ Replace the dust drawer until it clicks into place.
⇒ Press the enter key (25) (the counter which determines the time period is re-set).
- ♦ An acoustic signal confirms the input.
 - ♦ The display dust drawer (24) light disappears.

If the dust drawer is not emptied, the display for the dust drawer will remain alight (24). When the unit is switched off / on an acoustic signal will occur to remind that the dust drawer needs emptying.

For information on setting the required time period see the quick reference card which is supplied or go to the instructions for use in the Internet.

Change Fine Filter

- ▶ If the suction power is no longer sufficient in spite of a filter cleaning, the fine filter must be changed.
- ▶ In general, the fine filter should be changed every 2 years.

Do not clean the fine filter manually (e.g. with compressed air, brush, water, etc.), as this leads to damage of the filter material!

When installing the fine filter please ensure that it is positioned correctly, otherwise leakages may result. See the assembly instruction at the end of the instruction manual, which is also attached to the new fine filter.

EN

Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 29340000

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Disposal Information

Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local authorities for more information on proper disposal.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany